

AUTOMATIC LOADING STRETCHERS

1 Loading the stretcher

Reach the edge of the support with the stretcher's frontal loading wheels and push them on the platform.



Immediately after push the stretcher on the support, keeping the rear legs on the floor.



First hold the left safety lever, and then the handle on the right of the stretcher.



2 Locking the fixing system




Take the stretcher forward until the rear legs reach the ambulance.




At this point it is possible to push the stretcher onto the support until it locks thanks to the certified fixing system.

With the left hand grip now the handle on the left side of the stretcher and unlock the rear legs.

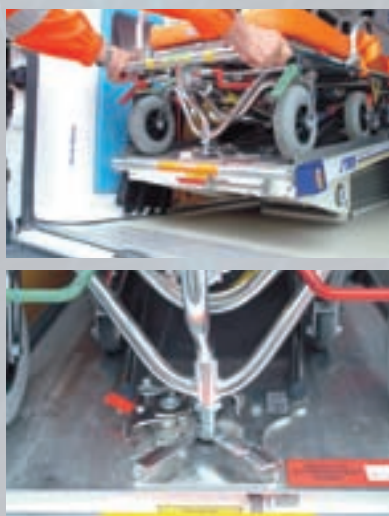

3 Unlocking the fixing system

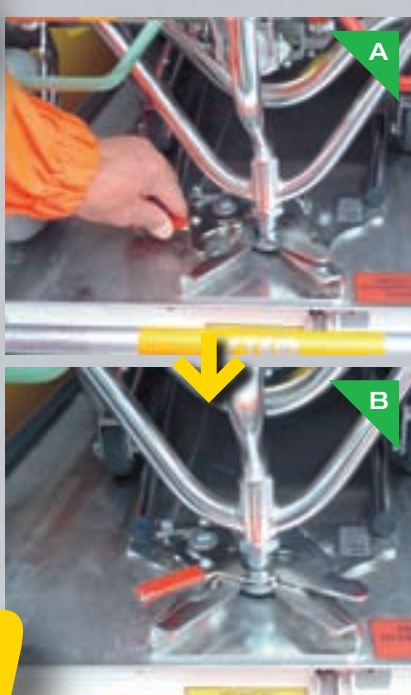

Rotate the stretcher-lock handle towards you, bringing it from position A to position B.



Now that the stretcher is unlocked, you can take it off the support.

It is important that the position of the hands is the same as shown in the figure.

4 Unloading the stretcher

Of primary importance is to assure that, during the unloading of the stretcher, there are at least 4/5 cm between the base of the wheel and the floor. Otherwise the front legs will not have enough space to open up, causing the accidental downfall of the stretcher.



SEDAN CHAIRS

1 Closing the sedan chairs



Lock the rotating wheels using the pedal brake.



Move to the side of the chair and hold the central levers with the right hand.



Lift the seat in a vertical position until the chair locks.




With the left hand, take the tube of the front part of the seat and, at the same time, squeeze and bring down the central unlocking levers.

2 Opening the sedan chairs



Move to the side of the chair. Hold the chair with one hand and with the other one grip the central levers. Squeeze and unlock the safety-catch.

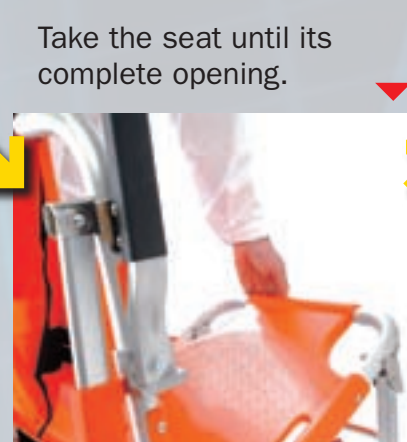

Take the seat until its complete opening.

